

COVID-19 Management

Perioperative Care

Optimizing N-95 Fit

- 1 After hand hygiene, remove glasses (if applicable).
 - 2 Cover nose and chin with N-95.
 - 3 Pull top strap over head above ears.
 - 4 Pull bottom strap over head below ears, ensuring bottom strap does not rest on hair.
 - 5 Untwist straps as needed.
 - 6 Press evenly and firmly to mold nosepiece.
- #### Seal Check

 - 7 Cup hand over N-95 edges.
 - 8 Inhale and exhale sharply.
 - 9 Readjust N-95 if leakage detected on edges.
 - 10 Repeat seal check as needed.
 - 11 Replace glasses and don eye protection.
 - 12 Repeat hand hygiene and don gloves.