


CARESCAPE B450 MONITOR REFERENCE SHEET

- Utilized for Blake 8 Cardiac SICU transports
- Touch screen
- Compatible brick with OR monitor
- Integrated AC/DC power supply unit and it supports two detachable lithium ion batteries

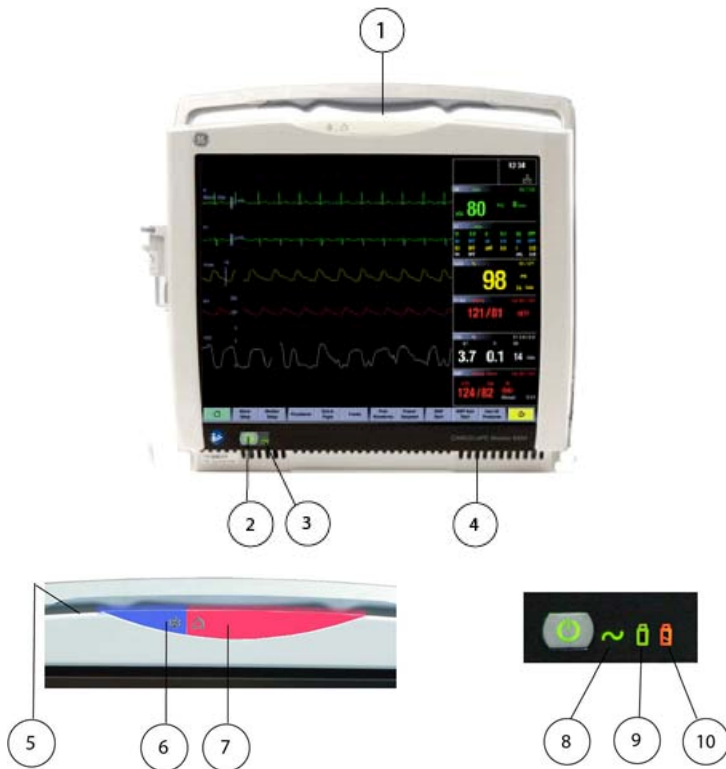


1. Alarm light
2. Power on/standby button
3. Power indicators
4. Ventilation holes
5. Ambient light detector lens
6. Audio alarm paused/off area (blue)
7. Alarm light area (blue, yellow, or red)
8. Mains voltage indicator – the green LED is lit when the patient monitor is connected to AC mains
9. Battery use indicator – the green light is lit when the patient monitor is operating on battery power
10. Battery charging/failure indicator – the orange LED is lit when the patient monitor battery is charging and flashing in case of battery failure or missing battery




CARESCAPE B450 MONITOR REFERENCE SHEET


Opening Screen:

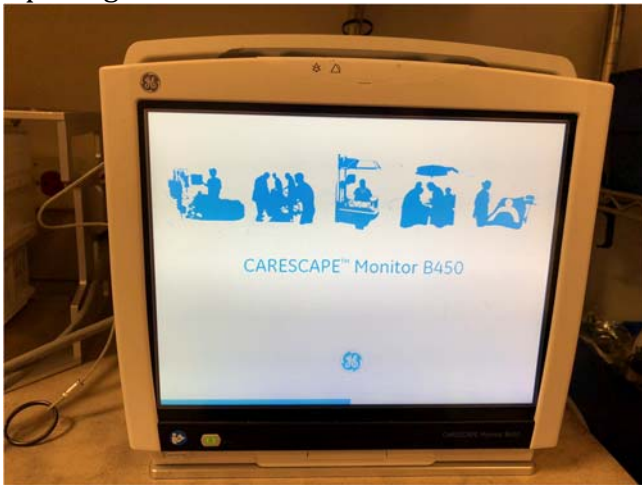

Select "Load PDM Data":

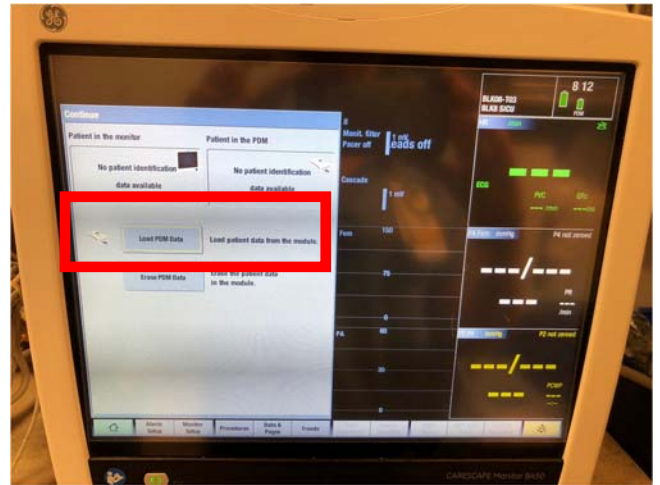


Zeroing Invasive Pressures:

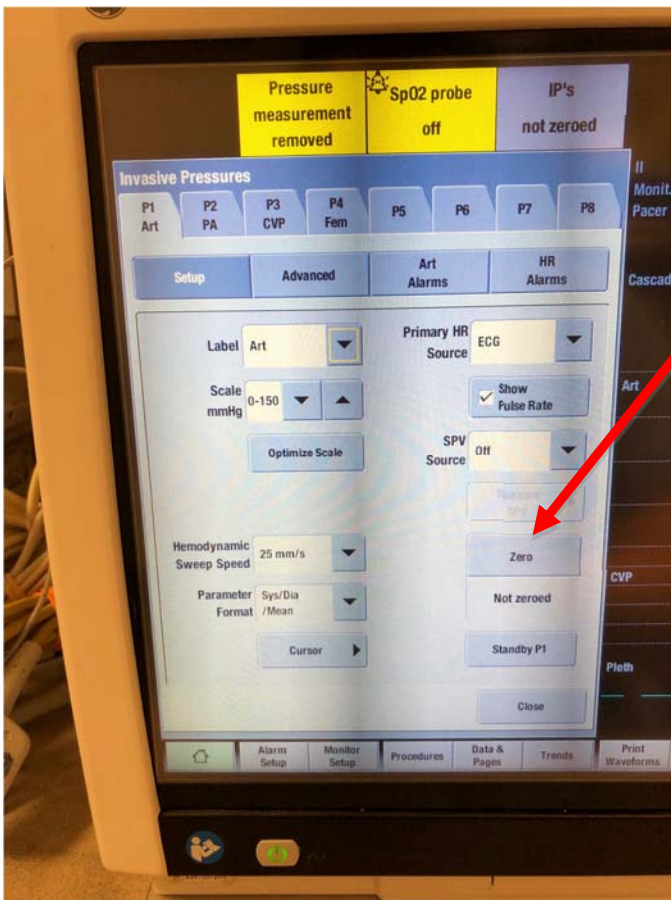

- Press the desired invasive pressure on the right side of the monitor's screen. Then press "Zero".



or

- Press "Zero All Pressures" tab.

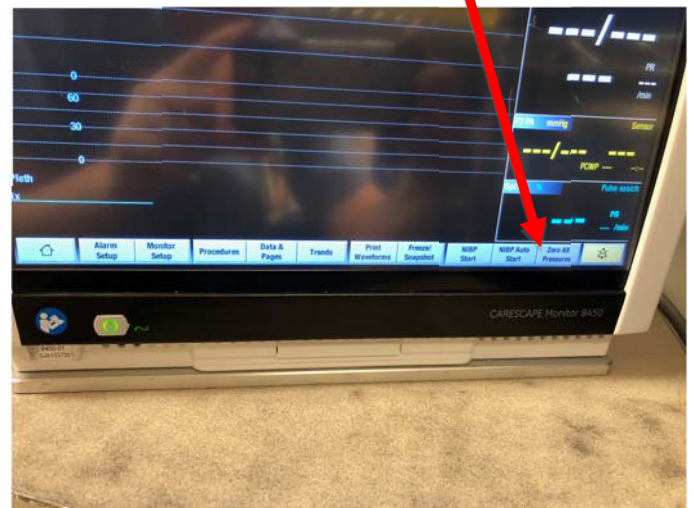


CARESCAPE B450 MONITOR REFERENCE SHEET

Alarm Silence:

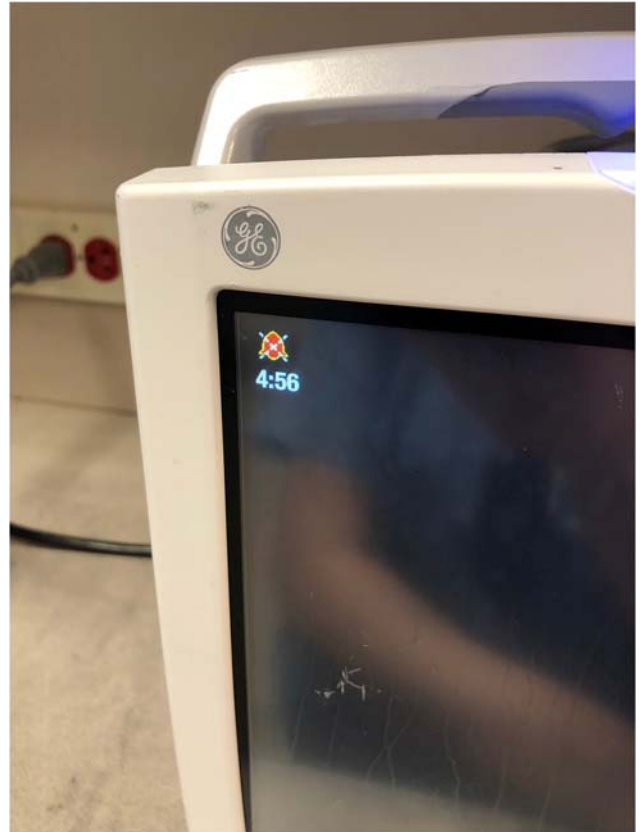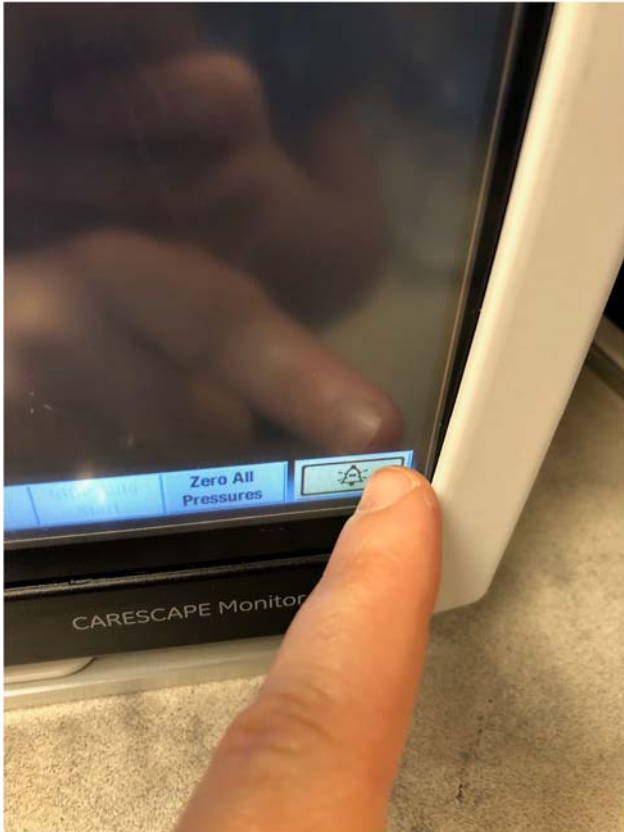


Powering On/Off:

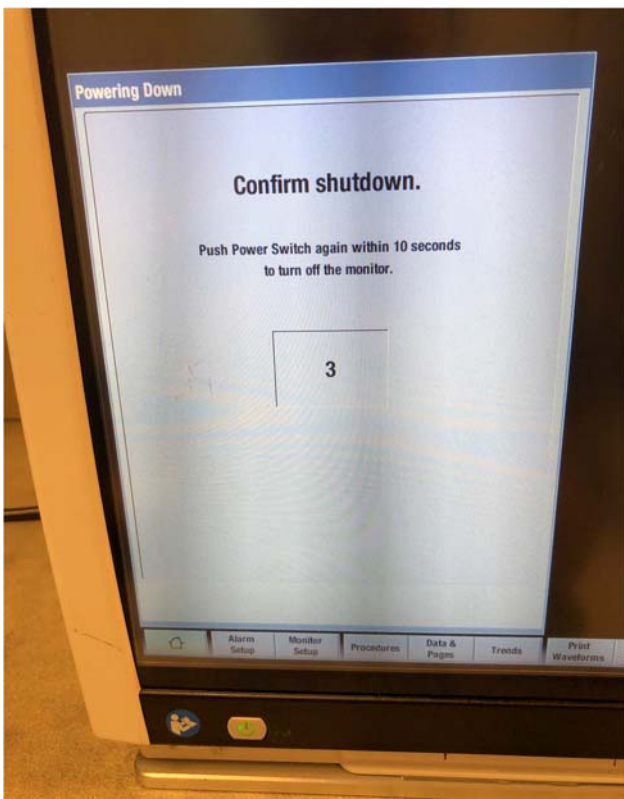


- Powering On: The monitor takes over a minute to turn on.
- Powering Off: The monitor requires pushing the power button twice.

CARESCAPE B450 MONITOR REFERENCE SHEET


Home Screen Tabs:
Alarm Setup

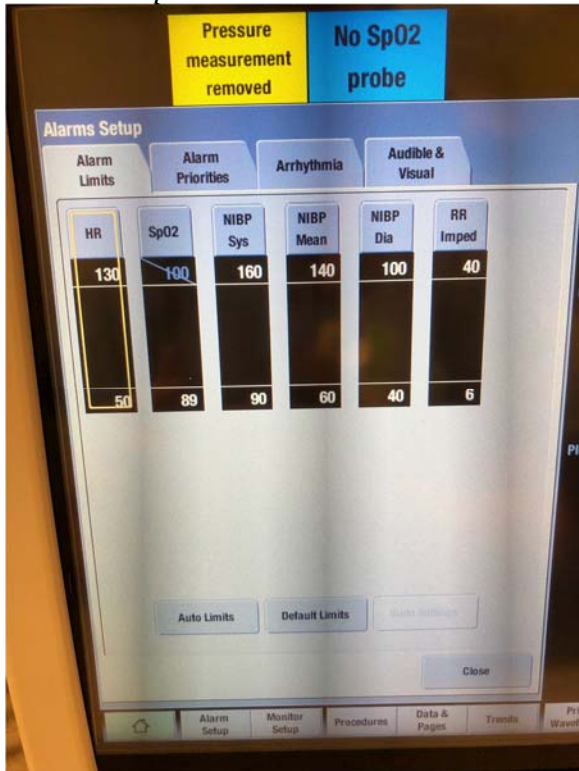


Monitor Setup

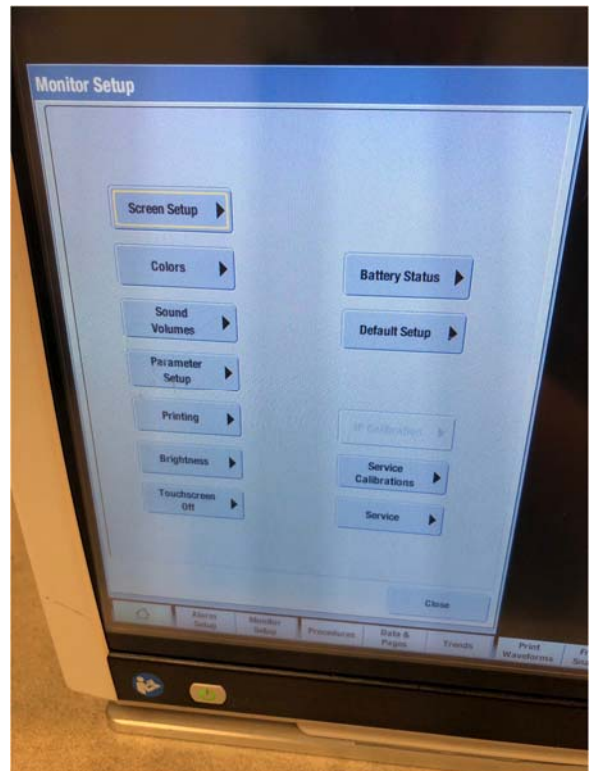

Procedures

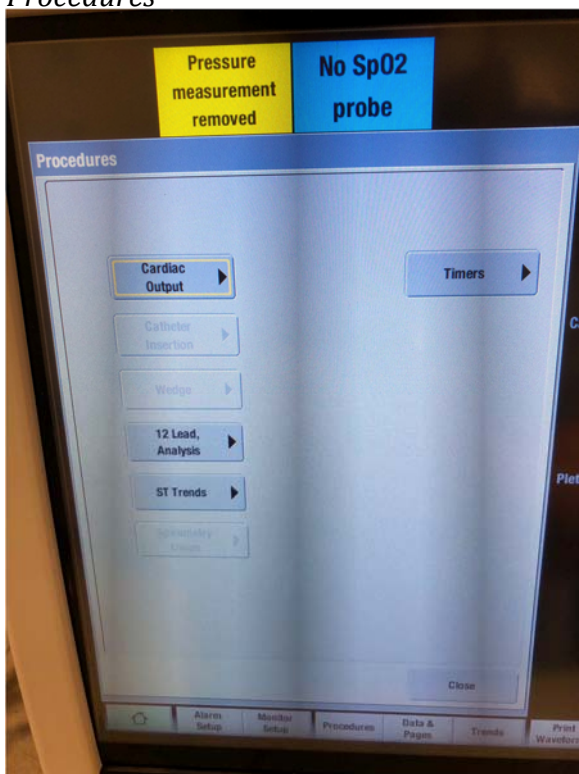


Data & Pages

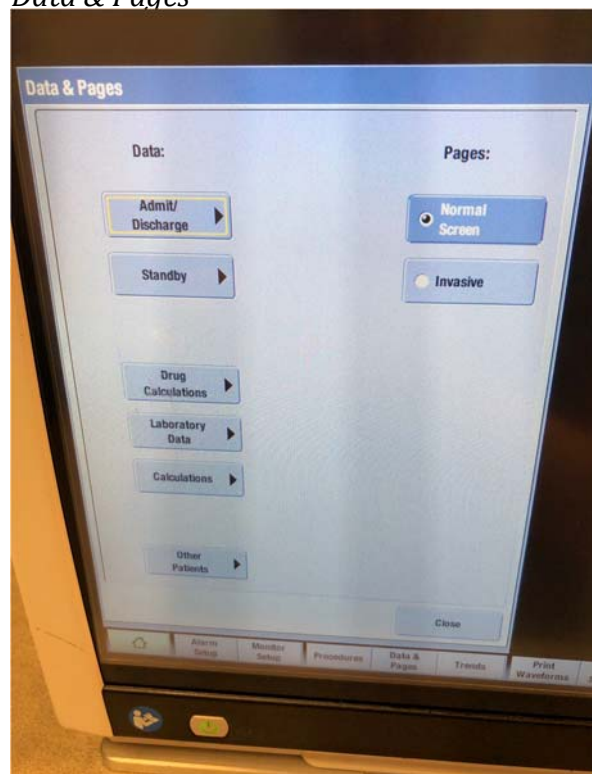

CARESCAPE B450 MONITOR REFERENCE SHEET

Procedures

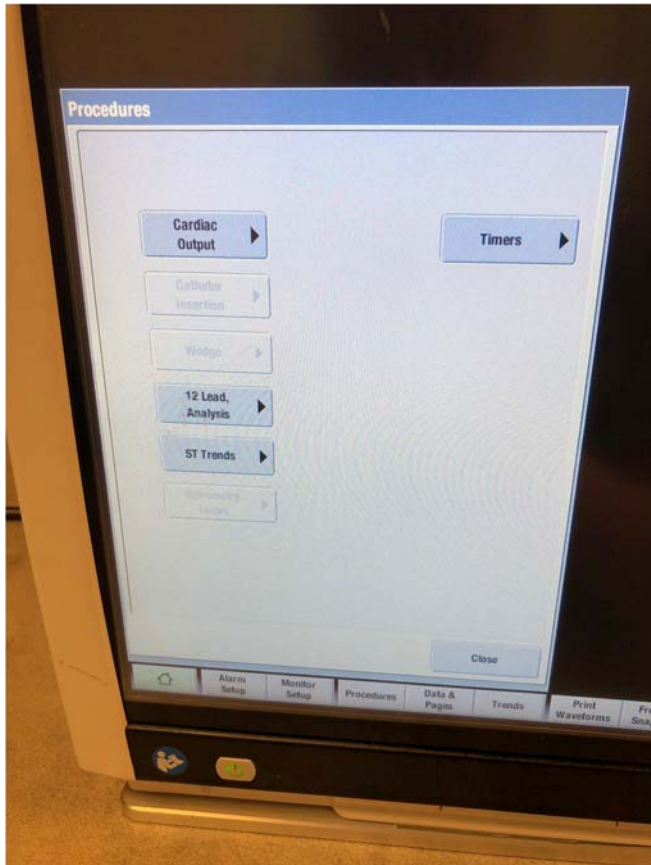

Trends

